



INSTRUCTIONS FOR SAMPLE COLLECTION

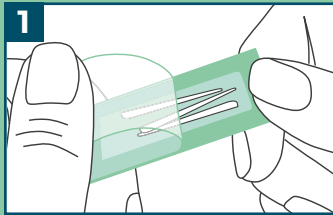

1. PREPARATION

Take the paper points and select the sampling site(s) for the samples.

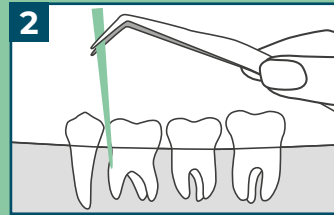


2. SAMPLING

Using sterile tweezers, insert a paper point down to the bottom of the dental pocket and leave it there for 10 to 15 seconds.



3. ORDER

Fill out the order form online; a paper form is optional; note the test number in your records. Tip: Take a photo of the batch label on the back of the test box.

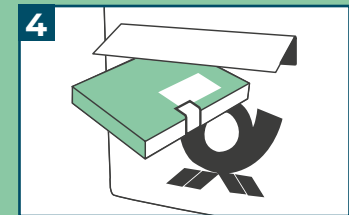

4. SHIPPING

Send in the samples and, if applicable documents postage paid in the PadoTest® kit box and obtain the result online and/or by mail/email.

BEFORE SAMPLING

- ▶ Remove supragingival plaque and dry the sampling site. (no professional tooth cleaning)
- ▶ Please note that extensive bleeding and/or pus discharge can falsify the analysis result.
- ▶ The use of antibacterial mouthwash solutions immediately before sampling should be avoided.

2A. MULTI-SITE TEST

Samples from multiple dental pockets are combined in a single analysis. Simply place all used paper points into one tube.

2B. SINGLE-SITE TEST

Separate analyses of samples from up to four different dental pockets. Each paper point is placed into a separate tube.

www.iai-test.de/order

ORDER ENTRY ONLINE

- ▶ Digital process
- ▶ Selection of control analyses
- ▶ Clinical details included in the results report
- ▶ Order copy via email

CONTENT:

order form | set of paper points | 4 tubes | security seal | instructions for use