



# QUICK GUIDE FOR SAMPLE COLLECTION

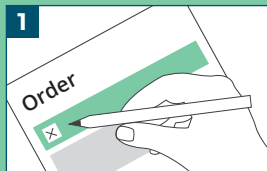

## 1. ORDER

Fill in all of the forms legibly using a pen.  
Note the test number of the form in your documents.

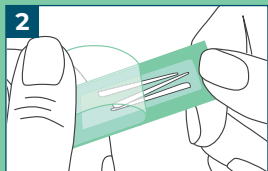

## 2. PREPARATION

Take the paper points and select the sampling point(s) for the samples.


### 2a. Multi-site test

Samples from several gingival sulci are summarised in one analysis.  
Simply place all of the paper points used into one test tube.

### 2b. Single-site test

Separate analyses of up to four different gingival sulci.  
Each paper point is supplied in a separate test tube.

*Prior to sampling: remove supragingival plaque and dry the sampling point (not professional tooth cleaning).*

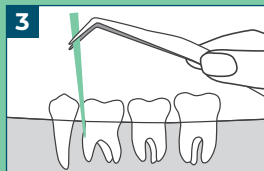


## 3. SAMPLING

Use sterile tweezers to insert a paper point down to the bottom of each gingival sulcus and leave it there for 10 to 15 seconds.



## 4. SUMMARISATION

Note the sampling point and pocket depth for the respective test tube on the order form.

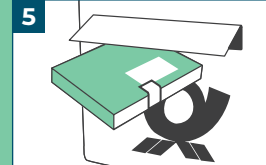

## 5. SHIPPING

Send in the samples and documents postage-paid in the **PadoTest®** kit box and obtain the result online and by post.

## CONTENTS

1 × order form, 1 × set of paper points,  
4 × test tubes, 1 × security seal,  
1 × quick guide